



G4 APP Download and Registration

Search APP Store

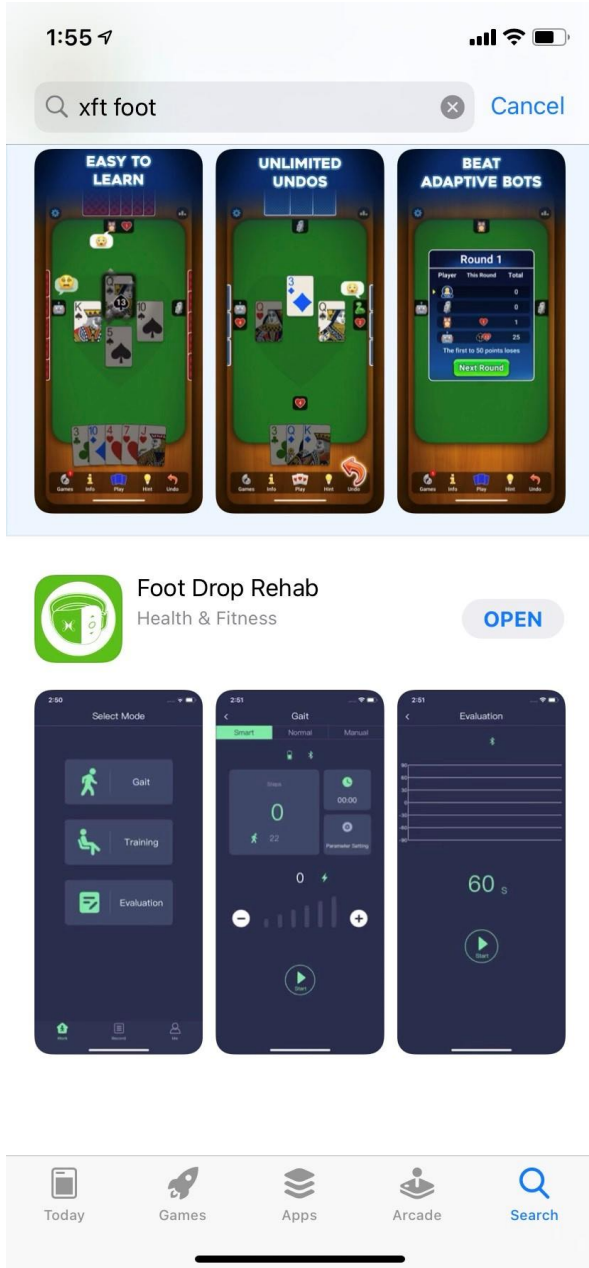

Download and open APP

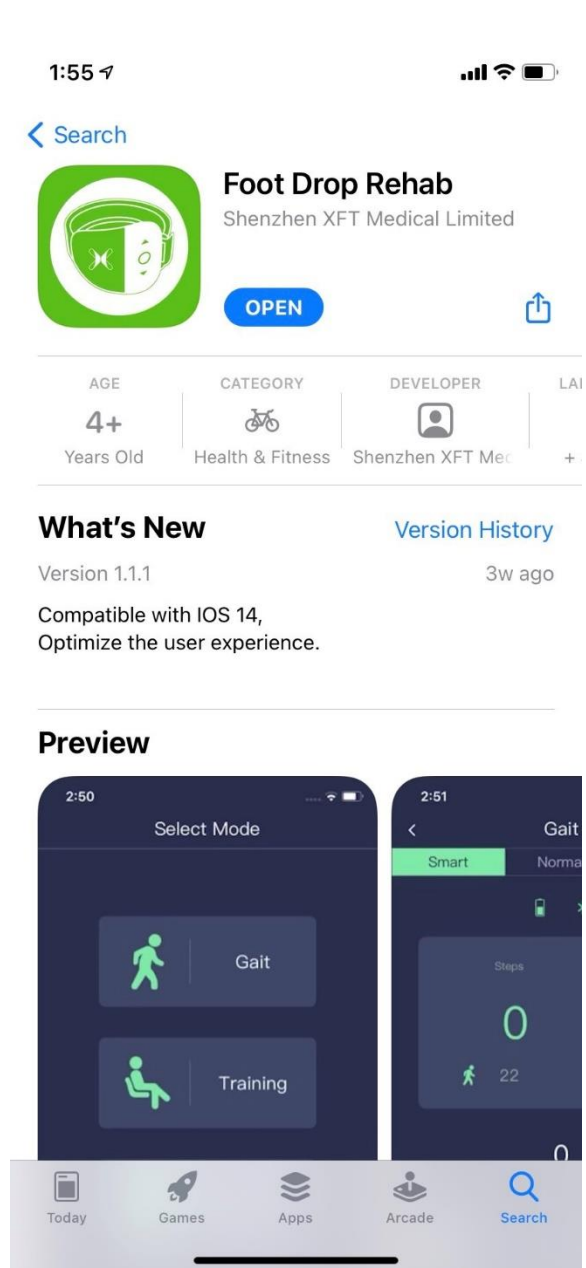

What's New

Version 1.1.1


Compatible with IOS 14,
Optimize the user experience.

Version History

3w ago

Preview

Choose America Dailing Code

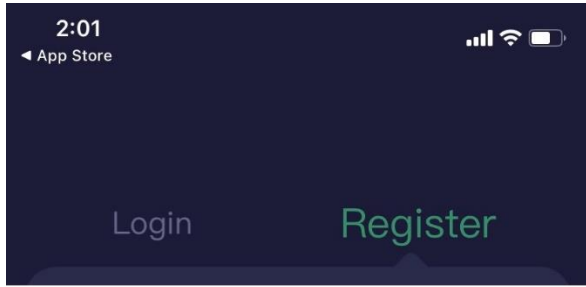

Cancel


A

Afghanistan	+93	
Albania	+335	
Algeria	+213	
America	+1	A
Andorra	+376	B
Angola	+244	C
Anguilla	+1254	D
Antigua and Barbuda	+1268	E
Argentina	+54	F
Armenia	+374	G
Ascension	+247	H
Australia	+61	I
Austria	+43	J
Azerbaijan	+994	K


B

— —


Enter Your Mobile Number. Get Verification Code

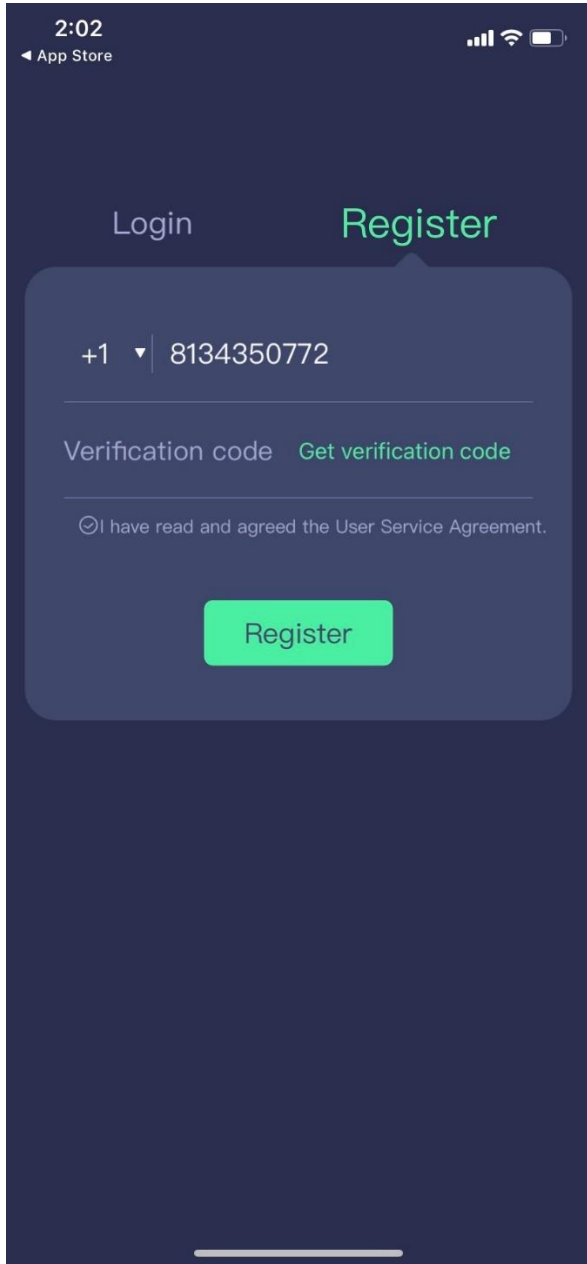


Get Verification Code

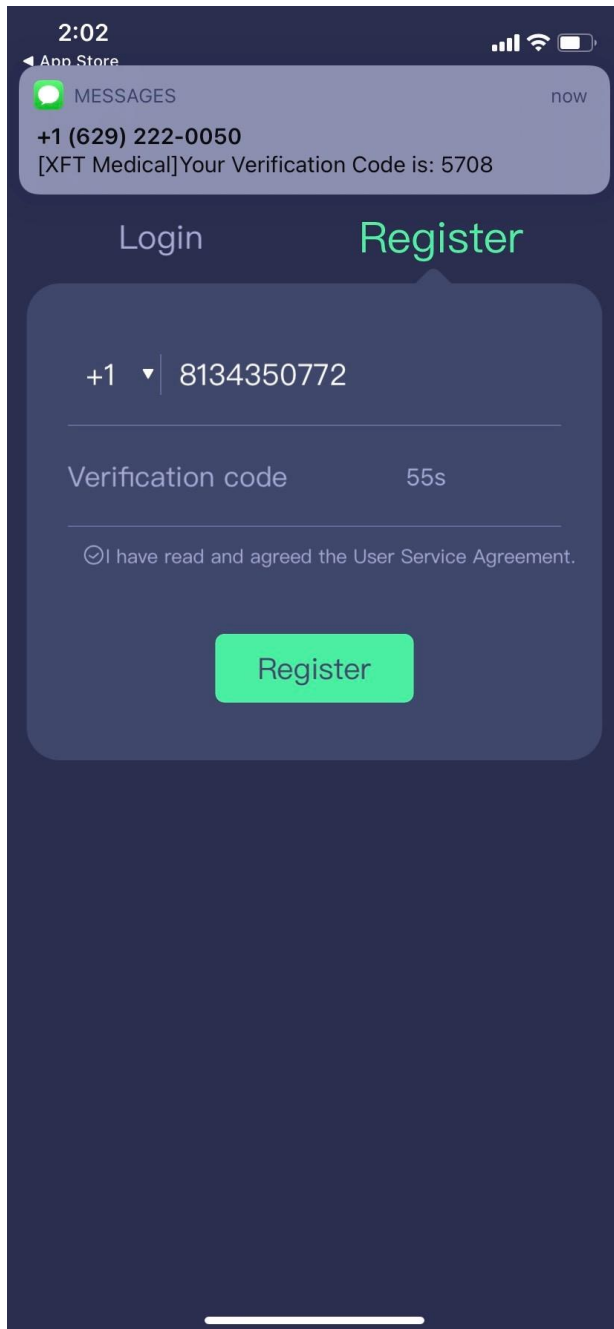


Type In Verification Code

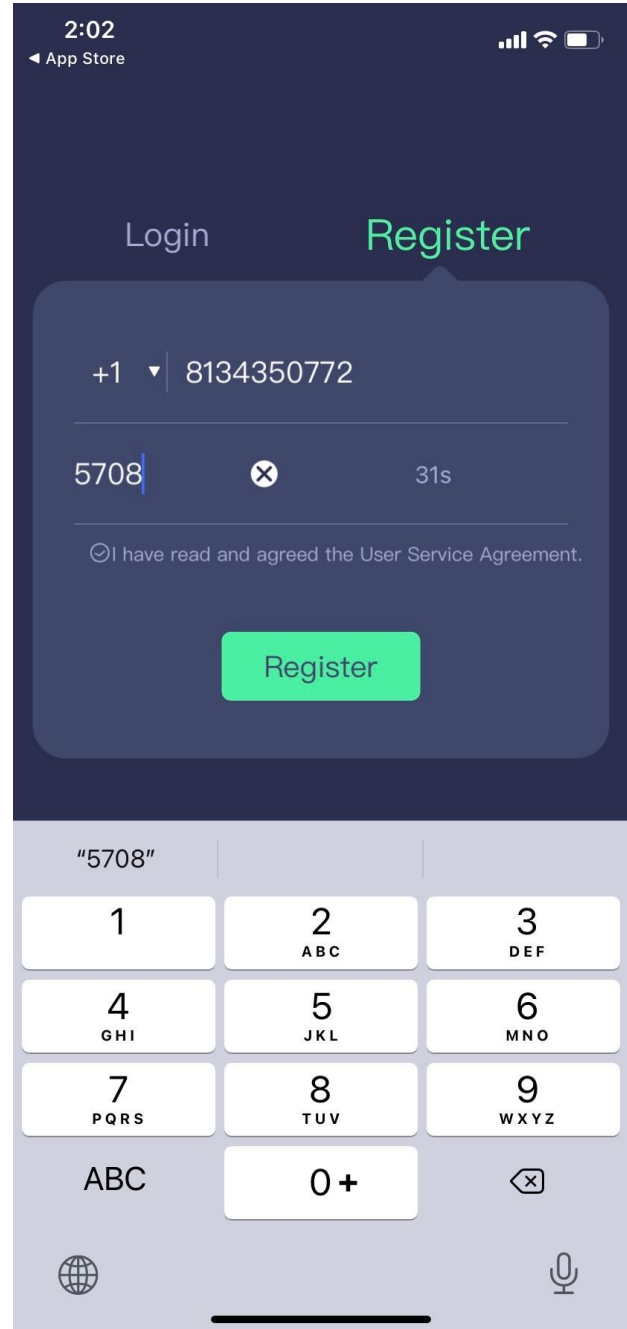

Finish Reigstration

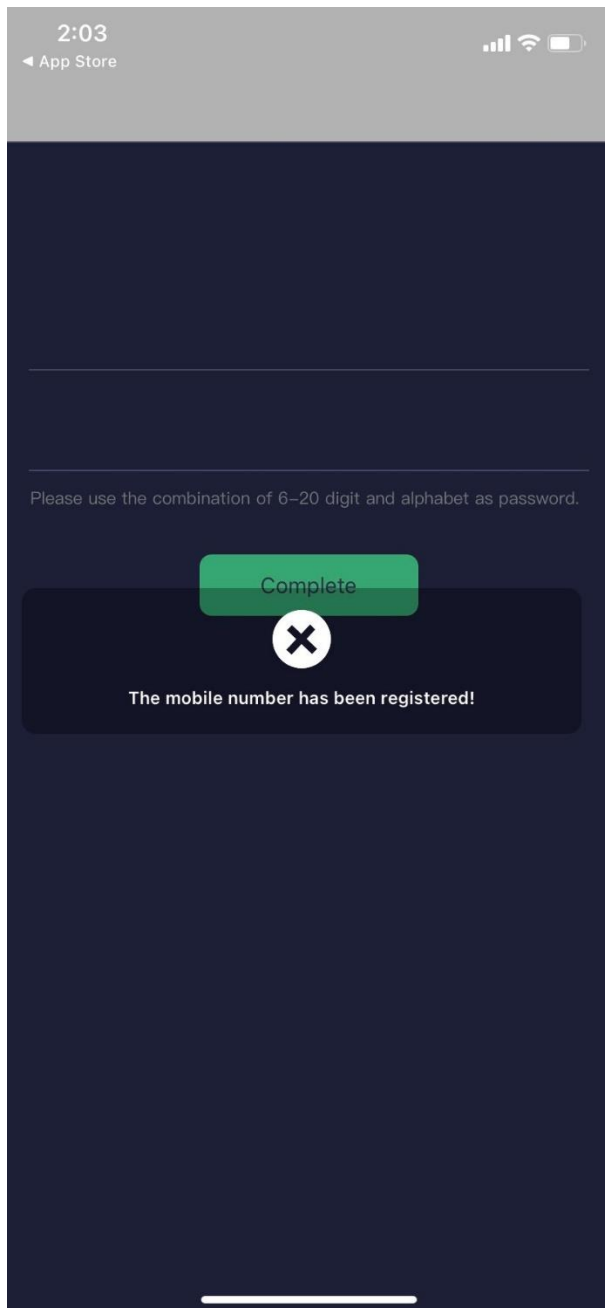

Set Password

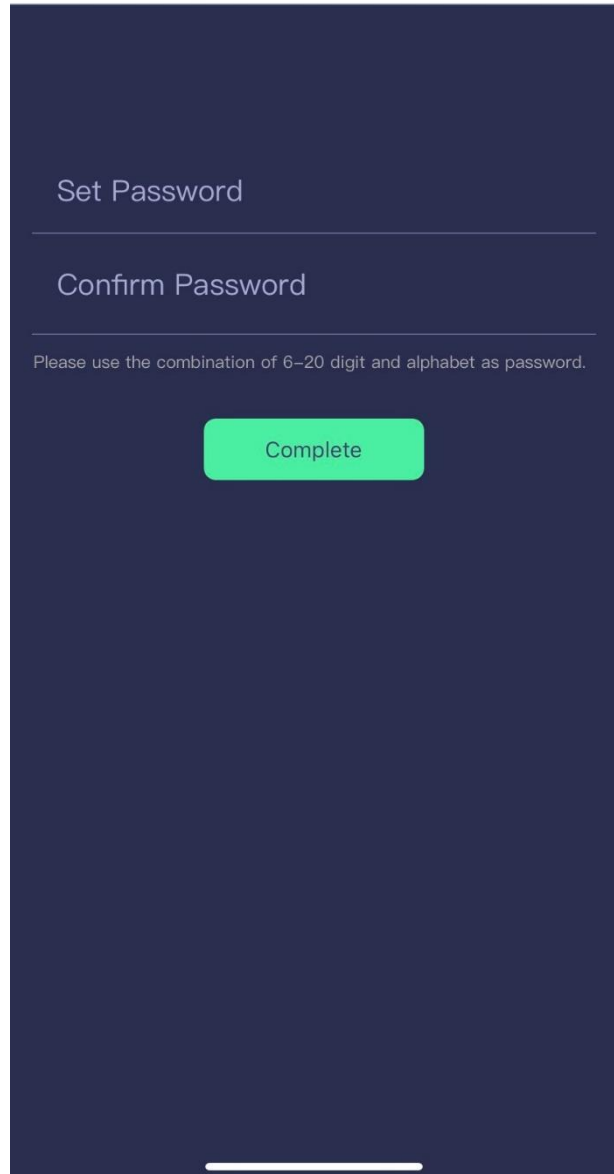

Enter Phone Number w/o Area Code and Password

2:07

Signal strength, Wi-Fi, and battery icons

Login Register

8134350722

x

Forget Password

Login

This will open the APP and take you to set up where you choose your language and pair to your device.

Make sure your phone blue tooth is on.

Make sure the WalkAide II is on.

It will search and then you can select and pair the device.

You will only have to do this once if you are using them same device.

To switch from the patient version to the Pro clinician version you will need to enter password: 846462